


The new
Cutera® Titan®
gives you an

all-natural option that
uses light energy to
heat the dermis deep
beneath your skin's
surface. Titan is an
ideal solution, without
surgery, recovery time,
or injections.

titan®

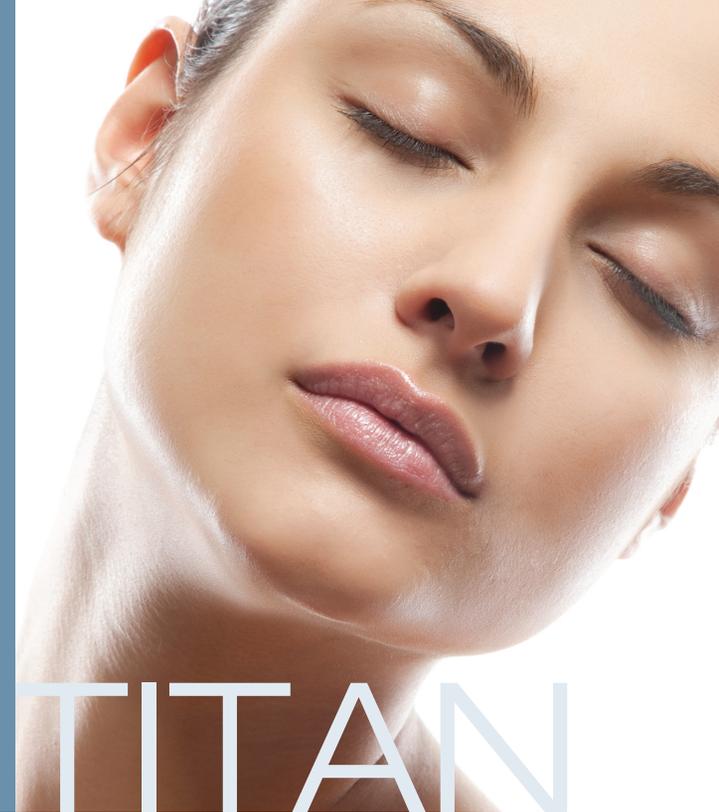

TITAN

Light-based
Treatment for
the Face & Body

CUTERA™

3240 Bayshore Boulevard, Brisbane, CA 94005
Phone: 1-888-4CUTERA | www.cutera.com

©2013 Cutera, Inc. All rights reserved. Titan and Cutera
are registered trademarks of Cutera, Inc.

AP0627 Rev A (6/13)

CUTERA™

Titan

How does the Cutera Titan work?

Titan utilizes a safe infrared light to heat the dermis well below the skin's surface. During the procedure, the skin's top surface (epidermis) is protected through continuous cooling with the Titan handpiece.

The Titan difference

Titan's light-based platform provides a more uniform distribution of energy for heating the deep dermis than other energy sources, such as RF (radio- frequency).

What areas respond best to the Titan procedure?

Patients have been successfully treated on many areas of the body. The most common areas treated are the jawline, under the chin, and the abdomen.

Who can be treated?

Since the Titan procedure heats tissue under the skin's surface, patients of any skin type or age can be treated. Your physician can advise you if the Titan is right for you.

How many treatments will I need?


Although a single treatment has produced very good results for many patients, it is quite common for patients to have two or three treatments over a few months. Your physician will advise you as to the number and frequency of treatments recommended for optimal results.

What happens after the treatment?

Immediately following the treatment, you may notice some redness and mild swelling in the treated area. Both should dissipate within a few hours. Consult with your physician for complete information on the benefits and risks of treatment.

What do treatments feel like?

With each Titan pulse you will feel a brief heating sensation. Some patients are more comfortable taking a mild pain reliever beforehand. Many patients are comfortable without any medication. Your physician will advise you.

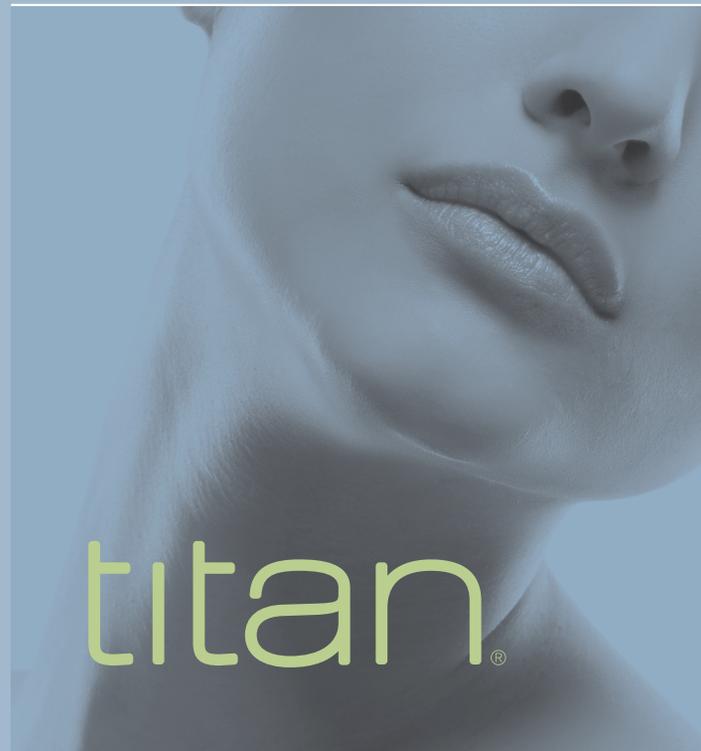

titan®

“You know you have a winner when patients keep calling to tell you about their great results. With no pain, no downtime, no anesthesia and plenty of results, it is becoming a popular part of my practice.”

Guido Lozada, M.D.
Plastic Surgeon

“Titan has quickly become a very popular procedure in my practice. Patient satisfaction has been very high with over sixty treated patients in just the past three months.”

Ronald Moy, M.D.
Dermatologist

Schedule your consultation now to learn more about the Titan procedure.

www.cutera.com